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Tel: +49 7142 99381-0 · Fax: +49 7142 99381-299

info@duerr-medical.de · www.duerr-medical.de

System requirements for computer systems in conjunction with DÜRR MEDICAL CR systems

The development of the operating systems is constantly being adapted to the steadily growing demands of the users. Today Windows XP and Windows Vista offer professional operating systems with sufficient multimedia functions. For reasons of system stability and data security, we recommend one of these operating systems.

If further systems are connected to the PC, the minimum system requirements may change. Observe the system requirements of all the interfaced systems.

Depending on the software used, higher demands may have to be made on the computer system.

1. System requirements

CPU:	Intel Pentium IV-compatible, 1.4 GHz or higher, no Celeron, no Duron
RAM:	min. 2 GB
Operating system:	single workstation system / multi-workstation system (network min. 100 MBit via switch): Microsoft Windows XP Professional, 32 bit, Service Pack 3 Microsoft Windows Vista, 32 bit, Service Pack 2 (from Home Premium)
Hard disk:	Workstation (without database) \geq 10 GB Database hard disk >100 GB

The memory requirements should be oriented to the number of images.

Drive:	DVD-ROM
Data backup:	Recommended for the daily data backup: Streamer (DAT, DITTO, TRAVAN, ADR (OnStream))

The database must be backed up with the daily data backup.

2. Appliance-specific system requirements

		CR 35 MED	CR 35 VET	CR 35 BIO	CR 43 MED	CR 43 VET	CR 7 VET	DR 6 VET
Mainboard (no SIS and ALI chip set):	Intel chip set / Via chip set	x	x	x	x	x	x	x
	Nvidia chip set / AMD chip set	x	x	x	x	x	x	x
Interface:	USB 2.0 interface	x	x	x			x	x
	Ethernet 100 Mbit				x	x	x	
Graphics board:	DirectX 9.0c with Direct Draw 9.0 enhancement or higher	x	x	x	x	x	x	x
	Resolution min. 1280 X 1024	x	x	x	x	x	x	x
	Depth of colour 32 bit, 16.7 million colours	x	x	x	x	x	x	x
	Memory \geq 128 MB	x	x	x	x	x	x	x
Special accessories:	Monitor according to X-Ray Ordinance, SVGA 17", \geq 1024 x 768 pixels, 24 / 32 bit depth of colour	x			x			
Maximum total cable length:	Network Cat5: 100 m				x	x	x	
	in addition 3x USB repeater (each max. 5 m) possible	x	x	x			x	x
	USB: 5 m	x	x	x			x	x

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DÜRR NDT GmbH & Co. KG